



Travel Cushion

Instructions



McFee Medical
TECHNOLOGIES

800.669.5323

Travel Cushion

Instructions

Position the Travel Cushion for lap use using the U-Bar with plastic knobs. QuickLock™ Face Rest and Crescent Face Cushion.

- 1 Slide the U-Bar into the pocket on the top of the Travel Cushion so that the chest plate Velcro is facing outward and the face rest brackets are at the top and inside.
- 2 Screw the plastic knobs into the Face Rest brackets loosely.
- 3 Fold the vinyl extension over the chest plate Velcro to hold the U-Bar in the Travel Cushion pocket and to prevent the Velcro from gripping on to clothing.
- 4 Place the QuickLock™ Face Rest into the brackets and the Travel Cushion on your lap with the chest plate towards your body.
- 5 To secure everything in position: move the Face Rest up and down, in and out, until it is in a comfortable and supportive position. Your eyes should be at the same level as your chin, or a little lower. When you have the Face Rest adjusted properly, use your hand to gently tighten the knobs and push down on the cam locks.