



**MajicView™**  
**Mirror**  
Instructions



**McFee Medical**  
T E C H N O L O G I E S

**800.669.5323**

# MajicView™ Mirror

## Instructions

Your MajicView™ is a patented device designed to make your recovery period a bit more tolerable, by enabling you to see the things normally while in a face down position. Here are few tips on how to use it. You will soon learn to depend on it for many ways improve your visual activities.

- 1 Place the MajicView™ directly below your eyes with the slanted side (side with the label) pointing inward, or towards you. Look down at the lower mirror. Your line of sight will be reflected upward into the other, and then outward in a near horizontal direction. The dual-mirror system produces an image that is upright, as in normal sight.
- 2 The closer the MajicView™ is to your face the broader the viewing area will be.
- 3 Moving the MajicView™ forward or backward will move the viewing area up or down. **NOTE:** Tilting the DEVICE ONLY does NOT change the viewing area; tilting BOTH the device AND your head together moves the images up or down.
- 4 You can rotate the MajicView™ to see objects to the left or right.